



## SPECIFICATIONS

Part No.	Alloy	Min Product Height	Max Product Height	Product Weight	Max Patient Weight	Set Screw Torque (M8x12mm)	Rotation Screw Torque (M8x6mm)
3T65	Titanium	51 mm.	57 mm.	158 gr.	136 Kg. / 300 lbs.	15 Nm.	9 Nm.
3T66	Titanium	58 mm.	64 mm.	172 gr.	136 Kg. / 300 lbs.	15 Nm.	9 Nm.

## USAGE

This product is designed to be used for lower limb external prosthetic length and alignment corrections. Before fitting this product onto the patient, all parts must be joined properly and all set screws must be secured enough for the patient to walk. In order to make final adjustments, there should be no loose screws.

## ATTENTION

After the final adjustments, apply single drop of removable Loctite® 242 to each of the set/cap screws and then torque the product. Improper installation or uneven, insufficient or excessive torque may cause bolt or adapter failure. Always use quality metric tools for installation.

## WARRANTY

All MEDEX Modular Adapters are unconditionally guaranteed for 2 years. Cutting, drilling, modifying or using the component other than as described in this manual will void the warranty.