



MEDEX INTERNATIONAL INC.

Torque Settings & Loctite® Recommendations

FOOT ADAPTERS

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
2T09	SACH FOOT ADAPTER (M8)	Titanium	M8 Bolt	1	15 Nm		80	100kg/220lbs
2T10	SACH FOOT ADAPTER (M10)	Titanium	M10x60 Bolt	1	20 Nm		93	136kg/300lbs
2S10	SACH FOOT ADAPTER (M10)	S.Steel	M10x60 Bolt	1	20 Nm		130	136kg/300lbs
2A10	SACH FOOT ADAPTER (M10)	Aluminum	M10x60 Bolt	1	20 Nm		106	136kg/300lbs
2T11	SACH FOOT ADAPTER (round)	Titanium	M10x60 Bolt	1	20 Nm		80	136kg/300lbs

PYLONS - CLAMPS

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
2A12	RECEIVER - TUBE UNIT	Aluminum	M8x12	4	15 Nm	✓	90	100kg/220lbs
2T12	RECEIVER - TUBE UNIT	Titanium	M8x12	4	15 Nm	✓	313	136kg/300lbs
2T17	EXTRA LONG TUBE WITH RECEIVER	Titanium	M8x12	4	15 Nm	✓	343	136kg/300lbs
2S17	EXTRA LONG TUBE WITH RECEIVER	S.Steel	M8x12	4	15 Nm	✓	385	136kg/300lbs
2T15	LONG TUBE WITH RECEIVER	Titanium	M8x12	4	15 Nm	✓	308	136kg/300lbs
2S15	LONG TUBE WITH RECEIVER	S.Steel	M8x12	4	15 Nm	✓	351	136kg/300lbs
2A15	LONG TUBE WITH RECEIVER	Aluminum	M8x12	4	15 Nm	✓	308	100kg/220lbs
2T16	SHORT TUBE WITH RECEIVER	Titanium	M8x12	4	15 Nm	✓	218	136kg/300lbs
2S16	SHORT TUBE WITH RECEIVER	S.Steel	M8x12	4	15 Nm	✓	261	136kg/300lbs
2A16	SHORT TUBE WITH RECEIVER	Aluminum	M8x12	4	15 Nm	✓	218	100kg/220lbs
2T13	HEAVY DUTY TUBE WITH RECEIVER	Titanium	M8x12	4	15 Nm	✓	334	205kg/450lbs
2T14	LONG TUBE WITH PYRAMID	Titanium	M8x12	4	15 Nm	✓	300	136kg/300lbs
2A18	SINGLE PYLON NO ADAPTER	Aluminum					242	136kg/300lbs
2T20	TUBE CLAMP ADAPTER	Titanium	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		70	136kg/300lbs
2S20	TUBE CLAMP ADAPTER	S.Steel	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		113	136kg/300lbs
2A20	TUBE CLAMP ADAPTER	Aluminum	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		68	100kg/220lbs
2T22	TUBE CLAMP ADAPTER (10° ANGLE)	Titanium	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		78	136kg/300lbs
4T55	ARCHED SLIDING TUBE CLAMP WITH THREADED ROTATION	Titanium	M5x12 - M5x16	2 - 1	9 Nm - 9 Nm		125	136kg/300lbs
2T23	HEAVY DUTY TUBE CLAMP RECEIVER	Titanium	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		75	205kg/450lbs
3T85	SLIDING TUBE CLAMP RECEIVER	Titanium	M8x12 - M5x12 - M5x16	4 - 2 - 1	15 Nm - 9 Nm - 9 Nm	M8X12	149	136kg/300lbs
3T87	SLIDING TUBE CLAMP RECEIVER ARC	Titanium	M8x12 - M5x12 - M5x16	4 - 2 - 1	15 Nm - 9 Nm - 9 Nm	M8X12	149	136kg/300lbs
3T86	SLIDING TUBE CLAMP PYRAMID	Titanium	M5x12 - M5x16	2 - 1	9 Nm - 9 Nm		158	136kg/300lbs
3T88	SLIDING TUBE CLAMP PYRAMID ARC	Titanium	M5x12 - M5x16	2 - 1	9 Nm - 9 Nm		158	136kg/300lbs

4 HOLE ADAPTERS

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
3T10	PYRAMID WITH 4 HOLES	Titanium					50	136kg/300lbs
3S10	PYRAMID WITH 4 HOLES	S.Steel					84	136kg/300lbs
3A10	PYRAMID WITH 4 HOLES	Aluminum					51	100kg/220lbs
3T09	PYRAMID WITH 4 HOLES & AXIAL OFFSET	Titanium					46	136kg/300lbs
3T11	PYRAMID WITH 4 HOLES & AXIAL OFFSET	Titanium					60	136kg/300lbs
3S11	PYRAMID WITH 4 HOLES & AXIAL OFFSET	S.Steel					108	136kg/300lbs
3T12	PYRAMID WITH 4 HOLES & DIAGONAL OFFSET	Titanium					60	136kg/300lbs
3S12	PYRAMID WITH 4 HOLES & DIAGONAL OFFSET	S.Steel					108	136kg/300lbs
3T18	PYRAMID ADAPTER WITH 4 HOLES	Titanium	M5x16	1	9 Nm		71	136kg/300lbs
3S18	PYRAMID ADAPTER WITH 4 HOLES	S.Steel	M5x16	1	9 Nm		124	136kg/300lbs
3A18	PYRAMID ADAPTER WITH 4 HOLES	Aluminum	M5x16	1	9 Nm		99	100kg/220lbs
3T19	PYRAMID ADAPTER WITH 4 HOLES (10mm holes)	Titanium	M5x16	1	9 Nm		71	136kg/300lbs
3T20	RECEIVER WITH 4 HOLES	Titanium	M8x12	4	15 Nm		65	136kg/300lbs
3S20	RECEIVER WITH 4 HOLES	S.Steel	M8x12	4	15 Nm		102	136kg/300lbs
3A20	RECEIVER WITH 4 HOLES	Aluminum	M8x12	4	15 Nm		40	100kg/220lbs
3T23	ROTATABLE RECEIVER WITH 4 HOLES	Titanium	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		80	136kg/300lbs
3S23	ROTATABLE RECEIVER WITH 4 HOLES	S.Steel	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		130	136kg/300lbs
3A23	ROTATABLE RECEIVER WITH 4 HOLES	Aluminum	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		74	100kg/220lbs
3T24	ROTATABLE PYRAMID ADAPTER W/4 HOLES & DIAGONAL OFFSET	Titanium	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		85	136kg/300lbs

3T25	ROTATABLE PYRAMID ADAPTER W/4 HOLES & AXIAL OFFSET	Titanium	M8x12 - M5x16	4 - 1	15 Nm - 9 Nm		85	136kg/300lbs
3T26	4 HOLE ROTATABLE FEMALE AXIAL OFFSET	Titanium	M8x12 - M5x12 - M5x16	4 - 2 - 1	15 Nm - 9 Nm - 9 Nm		114	136kg/300lbs
3T27	4 HOLE ROTATABLE FEMALE DIAGONAL OFFSET	Titanium	M8x12 - M5x12 - M5x16	4 - 2 - 1	15 Nm - 9 Nm - 9 Nm		114	136kg/300lbs
3T30	ROTATABLE SLIDING PYRAMID WITH 4 HOLES	Titanium	M5x12 - M5x16	2 - 1	9 Nm - 9 Nm		154	136kg/300lbs
3T31	ROTATABLE SLIDING PYRAMID WITH 4 HOLES ARC	Titanium	M5x12 - M5x16	2 - 1	9 Nm - 9 Nm		154	136kg/300lbs
3T35	ROTATABLE SLIDING RECEIVER WITH 4 HOLES	Titanium	M8x12 - M5x12 - M5x16	4 - 2 - 1	15 Nm - 9 Nm - 9 Nm		152	136kg/300lbs
3T36	ROTATABLE SLIDING RECEIVER WITH 4 HOLES ARC	Titanium	M8x12 - M5x12 - M5x16	4 - 2 - 1	15 Nm - 9 Nm - 9 Nm		152	136kg/300lbs
3T13	PYRAMID WITH 4 PRONGS	Titanium					62	136kg/300lbs
3T14	PYRAMID WITH 4 PRONGS	Titanium					52	136kg/300lbs
3S14	PYRAMID WITH 4 PRONGS	S.Steel					107	136kg/300lbs
3A14	PYRAMID WITH 4 PRONGS	Aluminum					52	100kg/220lbs
3T15	ROTATING PYRAMID WITH 4 PRONGS	Titanium	M8x6	4	9 Nm		92	136kg/300lbs
3S15	ROTATING PYRAMID WITH 4 PRONGS	S.Steel	M8x6	4	9 Nm		124	136kg/300lbs
3T17	4 SCREW PYRAMID WITH 4 PRONGS	Titanium	M8x12	4	15 Nm		56	136kg/300lbs
3S17	4 SCREW PYRAMID WITH 4 PRONGS	S.Steel	M8x12	4	15 Nm		92	136kg/300lbs
3T16	ROTATING RECEIVER WITH 4 PRONGS	Titanium	M8x12 - M8x6	4 - 3	15 Nm - 9 Nm		101	136kg/300lbs

DOUBLE HEAD PYRAMID

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
3T70	DOUBLE HEAD PYRAMID	Titanium					60	136kg/300lbs
3S70	DOUBLE HEAD PYRAMID	S.Steel					109	136kg/300lbs
3T60	DOUBLE HEAD RECEIVER AND PYRAMID	Titanium	M8x12	4	15 Nm	✓	76	136kg/300lbs
3S60	DOUBLE HEAD RECEIVER AND PYRAMID	S.Steel	M8x12	4	15 Nm	✓	124	136kg/300lbs
3A60	DOUBLE HEAD RECEIVER AND PYRAMID	Aluminum	M8x12	4	15 Nm	✓	65	100kg/220lbs
3T50	DOUBLE HEAD RECEIVER (32mm)	Titanium	M8x12	8	15 Nm		88	136kg/300lbs
3T51	DOUBLE HEAD RECEIVER (45mm)	Titanium	M8x12	8	15 Nm		98	136kg/300lbs
3T52	DOUBLE HEAD RECEIVER (60mm)	Titanium	M8x12	8	15 Nm		110	136kg/300lbs
3T53	DOUBLE HEAD RECEIVER (75mm)	Titanium	M8x12	8	15 Nm		120	136kg/300lbs
3S50	DOUBLE HEAD RECEIVER (32mm)	S.Steel	M8x12	8	15 Nm		135	136kg/300lbs
3S51	DOUBLE HEAD RECEIVER (45mm)	S.Steel	M8x12	8	15 Nm		156	136kg/300lbs
3S52	DOUBLE HEAD RECEIVER (60mm)	S.Steel	M8x12	8	15 Nm		179	136kg/300lbs
3S53	DOUBLE HEAD RECEIVER (75mm)	S.Steel	M8x12	8	15 Nm		197	136kg/300lbs
3A50	DOUBLE HEAD RECEIVER (32mm)	Aluminum	M8x12	8	15 Nm		65	100kg/220lbs
3A51	DOUBLE HEAD RECEIVER (45mm)	Aluminum	M8x12	8	15 Nm		80	100kg/220lbs
3A52	DOUBLE HEAD RECEIVER (60mm)	Aluminum	M8x12	8	15 Nm		94	100kg/220lbs
3A53	DOUBLE HEAD RECEIVER (75mm)	Aluminum	M8x12	8	15 Nm		112	100kg/220lbs

DOUBLE HEAD ROTATES AND SLIDES

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
3T54	DOAN'S DOUBLE HEAD ROTATABLE RECEIVER (34mm)	Titanium	M8x12 - M8x6	8 - 3	15 Nm - 9 Nm		114	136kg/300lbs
3T55	DOAN'S DOUBLE HEAD ROTATABLE RECEIVER (42mm)	Titanium	M8x12 - M8x6	8 - 3	15 Nm - 9 Nm		129	136kg/300lbs
3T56	DOAN'S DOUBLE HEAD ROTATABLE RECEIVER (53mm)	Titanium	M8x12 - M8x6	8 - 3	15 Nm - 9 Nm		143	136kg/300lbs
3T57	DOAN'S DOUBLE HEAD ROTATABLE RECEIVER (66mm)	Titanium	M8x12 - M8x6	8 - 3	15 Nm - 9 Nm		156	136kg/300lbs
3T61	DOUBLE HEAD ROTATABLE RECEIVER & PYRAMID (42mm)	Titanium	M8x12 - M8x6	4 - 3	15 Nm - 9 Nm		117	136kg/300lbs
3T63	DOUBLE HEAD ROTATABLE RECEIVER & PYRAMID (34mm)	Titanium	M8x12 - M8x6	4 - 3	15 Nm - 9 Nm		104	136kg/300lbs

ROTATES AND SLIDES

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
3T65	DOUBLE HEAD ROTATABLE ADAPTER WITH HEIGHT ADJUSTMENT (51-57 mm)	Titanium	M8x12 - M8x6	8 - 3	15 Nm - 9 Nm		158	136kg/300lbs
3T66	DOUBLE HEAD ROTATABLE ADAPTER WITH HEIGHT ADJUSTMENT (58-64 mm)	Titanium	M8x12 - M8x6	8 - 3	15 Nm - 9 Nm		172	136kg/300lbs
3T82	DOUBLE HEAD ROTATABLE SLIDING RECEIVER	Titanium	M8x12 - M8x6 - M5x12	8 - 3 - 2	15 Nm - 9 Nm - 9 Nm		192	136kg/300lbs
3T95	DOUBLE HEAD ROTATABLE SLIDING RECEIVER ARC	Titanium	M8x12 - M8x6 - M5x12	8 - 3 - 2	15 Nm - 9 Nm - 9 Nm		192	136kg/300lbs
3T83	DOUBLE HEAD ROTATABLE PYRAMID RECEIVER	Titanium	M8x12 - M8x6 - M5x12	4 - 3 - 2	15 Nm - 9 Nm - 9 Nm		193	136kg/300lbs
3T96	DOUBLE HEAD ROTATABLE PYRAMID RECEIVER ARC	Titanium	M8x12 - M8x6 - M5x12	4 - 3 - 2	15 Nm - 9 Nm - 9 Nm		193	136kg/300lbs
3T80	DOUBLE HEAD SLIDING RECEIVER	Titanium	M8x12 - M5x12	8 - 2	15 Nm - 9 Nm		157	136kg/300lbs

3T98	DOUBLE HEAD SLIDING RECEIVER ARC	Titanium	M8x12 - M5x12	8 - 2	15 Nm - 9 Nm		157	136kg/300lbs
3T81	DOUBLE HEAD SLIDING RECEIVER & PYRAMID	Titanium	M8x12 - M5x12	4 - 2	15 Nm - 9 Nm		157	136kg/300lbs
3T99	DOUBLE HEAD SLIDING RECEIVER & PYRAMID ARC	Titanium	M8x12 - M5x12	4 - 2	15 Nm - 9 Nm		157	136kg/300lbs

LAMINATION ANCHORS

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
4S20	3 ARM A/K LAMINATION ANCHOR WITH PYRAMID	S.Steel					125	136kg/300lbs
4S30	3 ARM A/K LAMINATION ANCHOR WITH ROTATABLE RECEIVER	S.Steel	M8x12 - M5x25	4 - 1	15 Nm - 9 Nm		184	136kg/300lbs
4ST6	3 ARM A/K LAMINATION ANCHOR WITH ROTATABLE RECEIVER	Titanium Head	M8x12 - M5x25	4 - 1	15 Nm - 9 Nm		160	136kg/300lbs
4S40	3 ARM A/K LAMINATION ANCHOR WITH ROTATABLE PYRAMID	S.Steel	M5x25	1	9 Nm		175	136kg/300lbs
4ST8	3 ARM A/K LAMINATION ANCHOR WITH ROTATABLE PYRAMID	Titanium Head	M5x25	1	9 Nm		150	136kg/300lbs
4T31	3 ARM A/K LAMINATION ANCHOR WITH ROTATABLE SLIDING RECEIVER	Titanium SS	M8x12 - M5x12 - M5x25	4 - 2 - 1	15 Nm - 9 Nm - 9 Nm		228	136kg/300lbs
4T41	3 ARM A/K LAMINATION ANCHOR WITH ROTATABLE SLIDING PYRAMID	Titanium SS	M5x12 - M5x25	2 - 1	9 Nm - 9 Nm		229	136kg/300lbs

LAMINATION PLATES

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
4TL1	LAMINATION PLATE WITH PYRAMID (OD: 65mm)	Titanium & Aluminum					95	136kg/300lbs
4TM1	LAMINATION PLATE WITH PYRAMID (OD: 55mm)	Titanium & Aluminum					77	136kg/300lbs
4TS1	LAMINATION PLATE WITH PYRAMID (OD: 45mm)	Titanium & Aluminum					62	136kg/300lbs
4TL2	LAMINATION PLATE WITH RECEIVER (OD: 65mm)	Titanium & Aluminum	M8x12	4	15 Nm		104	136kg/300lbs
4TM2	LAMINATION PLATE WITH RECEIVER (OD: 55mm)	Titanium & Aluminum	M8x12	4	15 Nm		86	136kg/300lbs
4TS2	LAMINATION PLATE WITH RECEIVER (OD: 45mm)	Titanium & Aluminum	M8x12	4	15 Nm		72	136kg/300lbs

SHUTTLE LOCKS

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
4A25	4 SCREW MOUNTING SHUTTLE LOCK	Aluminum	M10 Pin	1			99	100kg/220lbs
4T28	4 PRONG COMBINATION SHUTTLE LOCK	Titanium	M10 Pin - M8x6	1 - 3	0 - 9 Nm		147	136kg/300lbs
4T26	SCREW THREAD SHUTTLE LOCK	Titanium	M10 Pin - M5x25	1 - 1	0 - 9 Nm		220	136kg/300lbs
4T25	4 SCREW MOUNTING SHUTTLE LOCK	Titanium	M10 Pin - M5x16	1 - 1	0 - 9 Nm		184	136kg/300lbs

RIM SYSTEMS

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
4AT3	LAMINATION RIM WITH ROTATABLE PYRAMID (OD: 70mm)	Titanium & Aluminum	M8x12	3	9 Nm		122	136kg/300lbs
4AT4	LAMINATION RIM WITH ROTATABLE PYRAMID (OD: 63mm)	Titanium & Aluminum	M8x6	3	9 Nm		93	136kg/300lbs
4AT5	LAMINATION RIM WITH ROTATABLE SLIDING PYRAMID (OD: 70mm)	Titanium & Aluminum	M8x12 - M5x12	3 - 2	9 Nm - 9 Nm		197	136kg/300lbs
4AT6	LAMINATION RIM WITH ROTATABLE SLIDING PYRAMID (OD: 63mm)	Titanium & Aluminum	M8x6 - M5x12	3 - 2	9 Nm - 9 Nm		183	136kg/300lbs
4AT1	LAMINATION RIM WITH ROTATABLE RECEIVER (OD: 70mm)	Titanium & Aluminum	M8x12	7	9 Nm		117	136kg/300lbs
4AT2	LAMINATION RIM WITH ROTATABLE RECEIVER (OD: 63mm)	Titanium & Aluminum	M8x12 - M8x6	4 - 3	9 Nm - 9 Nm		99	136kg/300lbs
4AT7	LAMINATION RIM WITH ROTATABLE SLIDING RECEIVER (OD: 70mm)	Titanium & Aluminum	M8x12 - M5x12	7 - 2	9 Nm - 9 Nm		196	136kg/300lbs
4AT8	LAMINATION RIM WITH ROTATABLE SLIDING RECEIVER (OD: 63mm)	Titanium & Aluminum	M8x12 - M8x6 - M5x12	4 - 3 - 2	9 Nm - 9 Nm - 9 Nm		178	136kg/300lbs
4A20	RIM SYSTEM SHUTTLE LOCK ALUMINUM (OD: 63MM) WATERPROOF	Aluminum & S. Steel	M8x6	3	9 Nm		146	100kg/220lbs
4T20	RIM SYSTEM SHUTTLE LOCKS TITANIUM (OD: 70mm)	Titanium & Aluminum	M8x12	3	9 Nm		200	136kg/300lbs
4T21	RIM SYSTEM SHUTTLE LOCKS TITANIUM (OD: 63mm)	Titanium & Aluminum	M8x6	3	9 Nm		181	136kg/300lbs

KNEE JOINTS

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
5T40	MODULAR SINGLE AXIS KNEE JOINT W/LOCK	Titanium					349	100kg/220lbs
5S40	MODULAR SINGLE AXIS KNEE JOINT W/LOCK	S.Steel					525	100kg/220lbs
5T50	MODULAR SINGLE AXIS KNEE JOINT	Titanium					385	100kg/220lbs
5S50	MODULAR SINGLE AXIS KNEE JOINT	S.Steel					525	100kg/220lbs

ROTATING TIPS AND FRAMES

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
4T09	ROTATING PYRAMID TIP	Titanium					38	136kg/300lbs
4T10	ROTATING PYRAMID TIP 2mm longer	Titanium					44	136kg/300lbs
4T11	ROTATING PYRAMID TIP 4mm longer	Titanium					48	136kg/300lbs
4T08	ROTATING RECEIVER TIP	Titanium	M8x12	4	15 Nm		48	136kg/300lbs

4T14	ROTATABLE SLIDING PYRAMID TIP	Titanium	M5x12	2	9 Nm		122	136kg/300lbs
4T50	ROTATABLE SLIDING PYRAMID TIP ARC	Titanium	M5x12	2	9 Nm		122	136kg/300lbs
4T15	ROTATABLE SLIDING RECEIVER TIP	Titanium	M8x12 - M5x12	4 - 2	15 Nm - 9 Nm		120	136kg/300lbs
4T51	ROTATABLE SLIDING RECEIVER TIP ARC	Titanium	M8x12 - M5x12	4 - 2	15 Nm - 9 Nm		120	136kg/300lbs
3T90	4 HOLE ROTATING FRAME ONLY	Titanium	M5x16	1	9 Nm		33	136kg/300lbs
4S35	3 ARM LAMINATION ANCHOR ONLY	S.Steel	M5x25	1	9 Nm		109	136kg/300lbs
4A80	CONNECTOR PLATE	Aluminum					70	136kg/300lbs
4A65	COUPLING (24mm)	Aluminum					75	136kg/300lbs
4A70	COUPLING (17mm)	Aluminum					45	136kg/300lbs

PEDIATRIC COMPONENTS

Catalog Number	Description	Material	Screw(s)	Qty	Torque (Nm)	Loctite	Weight (gr)	Max Patient Weight
2CT5	CHILD SACH FOOT ADAPTER	Titanium					62	80kg/175lbs
2CT6	CHILD PYLON WITH RECEIVER	Titanium				v	113	80kg/175lbs
2CT7	CHILD TUBE CLAMP ADAPTER	Titanium	M6x10 - M4x10	4 - 1	12 Nm - 9 Nm		40	80kg/175lbs
3CT5	CHILD PYRAMID WITH 4 HOLES	Titanium					26	80kg/175lbs
4CT9	CHILD SLIDING TUBE CLAMP PYRAMID	Titanium	M6x10 - M4x10 - M5x10	4 - 1 - 2	12 Nm - 9 Nm - 10 Nm		93	80kg/175lbs
3CT1	CHILD DOUBLE HEAD ROTATABLE RECEIVER (26mm)	Titanium	M6x10 - M6x6	8 - 3	12 Nm - 9 Nm		69	80kg/175lbs
3CT2	CHILD DOUBLE HEAD ROTATABLE RECEIVER (37mm)	Titanium	M6x10 - M6x6	8 - 3	12 Nm - 9 Nm		79	80kg/175lbs
3CT3	CHILD DOUBLE HEAD ROTATABLE RECEIVER (48mm)	Titanium	M6x10 - M6x6	8 - 3	12 Nm - 9 Nm		88	80kg/175lbs
3CT4	CHILD DOUBLE HEAD ROTATABLE RECEIVER (60mm)	Titanium	M6x10 - M6x6	8 - 3	12 Nm - 9 Nm		94	80kg/175lbs
3CT6	CHILD DOUBLE HEAD RECEIVER (22mm)	Titanium	M6x10	8	12 Nm		44	80kg/175lbs
3CT7	CHILD DOUBLE HEAD RECEIVER (32mm)	Titanium	M6x10	8	12 Nm		51	80kg/175lbs
3CT8	CHILD DOUBLE HEAD RECEIVER (45mm)	Titanium	M6x10	8	12 Nm		60	80kg/175lbs
3CT9	CHILD DOUBLE HEAD RECEIVER (60mm)	Titanium	M6x10	8	12 Nm		70	80kg/175lbs
3TC1	CHILD ROTATING PYRAMID WITH 4 HOLES	Titanium	M4x10	1	9 Nm		44	80kg/175lbs
3TC2	CHILD ROTATING RECEIVER WITH 4 HOLES	Titanium	M6x10 - M4x10	4 - 1	12 Nm - 9 Nm		51	80kg/175lbs
4CT1	CHILD DOUBLE HEAD ROTATABLE RECEIVER & PYRAMID	Titanium	M6x10 - M6x6	4 - 3	12 Nm - 9 Nm		66	80kg/175lbs
4CT2	CHILD DOUBLE HEAD ROTATABLE RECEIVER & PYRAMID	Titanium	M6x10 - M6x6	4 - 3	12 Nm - 9 Nm		72	80kg/175lbs
4CT3	DOUBLE HEAD ADULT PYRAMID TO PEDIATRIC CONVERTER	Titanium	M8x12	4	15 Nm		68	80kg/175lbs
4CT7	CHILD DOUBLE HEAD ARCHED SLIDING RECEIVER	Titanium	M6x10 - M5x10	8 - 2	12 Nm - 10 Nm		116	80kg/175lbs
4CT8	CHILD DOUBLE HEAD ARCHED SLIDING RECEIVER & PYRAMID	Titanium	M6x10 - M5x10	4 - 2	12 Nm - 10 Nm		121	80kg/175lbs
4TC1	CHILD 3 ARM LAMINATION ANCHOR WITH ROTATABLE RECEIVER	Titanium	M6x10 - M6x6	4 - 3	12 Nm - 9 Nm		116	80kg/175lbs
4TC2	CHILD 3 ARM LAMINATION ANCHOR WITH ROTATABLE PYRAMID	Titanium	M6x6	3	9 Nm		108	80kg/175lbs
4TC3	CHILD 4 ARM LAMINATION ANCHOR WITH ROTATABLE RECEIVER	Titanium	M6x10 - M6x6	4 - 3	12 Nm - 9 Nm		126	80kg/175lbs
4TC4	CHILD 4 ARM LAMINATION ANCHOR WITH ROTATABLE PYRAMID	Titanium	M6x6	3	9 Nm		119	80kg/175lbs
4TC5	CHILD LAMINATION RIM WITH ROTATING PYRAMID	Titanium	M6x6	3	9 Nm		66	80kg/175lbs
4TC6	CHILD LAMINATION RIM WITH ROTATING RECEIVER	Titanium	M6x10 - M6x6	4 - 3	12 Nm - 9 Nm		72	80kg/175lbs
4TC7	CHILD CONNECTOR PLATE	Titanium					116	80kg/175lbs