**Technical Specification**

Part No.	4T25	4A25
Alloy	Titanium	Aluminium
Product Weight	184 gr.	99 gr.
Product Height	29 mm.	29 mm.
Max Patient	136 kg. / 300	100 kg. / 220
Torque	9 Nm	9 Nm

**Usage**

Cast preparation and Dummy installation:

- After all alteration is done on the cast, the distal end of the socket must be checked with the dummy to ensure proper fit; make necessary alterations to the cast until fit is exact. The 4 holes on the dummy should be plugged, filled or taped to keep clean for future use.
- Once dummy properly fits to distal end, ensure desired alignment and right location, then nail/attach it from center hole. Fill up the joint area between the cast and dummy with plaster of paris to make sure smooth contact is achieved between dummy and cast.
- Once the cast is ready and dummy is in place for lamination, put PVA bag over the cast and dummy. While the PVA bag is soft and loose, tie the inner disk (small circumference) of the dummy securely and then fasten the PVA bag to top and bottom of the cast. Prepare to apply suction.
- To increase suction quality and the imprint of dummy, use a knee-high stockinet under the PVA bag.
- Second option for dummy installation is over the PVA bag. This method allows PVA bag to go first, then the dummy is installed externally over the PVA bag. PVA bag can be taped securely at the wider circumference of the dummy and lamination can be applied directly on the dummy. Be sure to get the best possible imprint of the dummy inside of the socket. After dummy installation over the PVA bag, reinforcement layers can be applied with strong suction.

**Assembly of Shuttle Lock:**

- After removing socket from cast, smooth all the edges. Tap lightly on the dummy from the bottom of the socket, outside of distal end to remove it. Remove plug or tape from the 4 holes of the dummy. Reverse dummy and install the smaller diameter end into distal end of socket hole.
- The 4 hole pattern of the dummy will be used as a guide for marking and drilling 4 holes on the socket for inner connection ring. As long as the rotatable type shuttle lock is used (4T25/4A25), angle of the holes is not very critical. You can adjust rotation later.
- After the 4 holes on the socket have been drilled, you will no longer need the dummy. Now check to see if socket's distal center hole is clean enough to receive shuttle locks inner retainer part.
- Check distal end of the socket to make sure its level and even thickness all around. Then install retainer inside of the socket and make sure retainer screw holes match the holes previously drilled on the socket. Use mounting screws to attach aluminum ring.
- Screws that are supplied with the shuttle lock are intended for socket wall thickness of approximately 6mm. Socket wall thickness of 5mm or less needs shorter screws, otherwise it may crack inner pin guide cup. If the thickness exceeds 7mm, then supplied screws may be too short to go all the way through the threads of retainer ring. Please make sure that appropriate mounting screw lengths are used.
- Now try liner with pin or just the pin itself to see how Shuttle Lock performs.

**Maintenance**

Maintenance is to be carried out by competent personnel only. An annual visual inspection is recommended. During maintenance, it is appropriate to consider any weight or activity increase by the user. If the prosthesis is exposed to circumstances or impact for which it was not designed, inspection by a qualified practitioner is recommended.

- Ensure all screws are secure. If not, remove and clean screws, reapply Loctite to the pyramid screws only, tighten all screws to the correct torque settings.
- Check for visual defects that may affect proper function.

The user should be advised to contact their practitioner if:

- Their condition changes.
- Performance of the device changes.
- Changes in performance may include:
  - Any unusual noises
  - Excessive play or loss of alignment

**Warranty**

All MEDEX Modular Adapters are unconditionally guaranteed for 2 years. Cutting, drilling, modifying or using the component other than as described in this manual will void the warranty.

**Liability**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device.

**CE Conformity**

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines.

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